

Reversible SlantRail™


Item# 600




The Reversible SlantRail™ is a unique Bed Assist Handle that provides two different handles in one. Depending on your needs, this handle provides a wide range of grasp and leverage positions to help promote better bed mobility. Two tubular support tubes slide between the mattress and box spring. Each support tube has a foam wrap to prevent the SlantRail from sliding.

Assembly Instructions:

1. Slide mattress down on bed. To assemble the Reversible SlantRail, lay the unit on top of the Boxspring. Insert the support tubes into handle legs. Secure in place with snap buttons. (figure 1 & figure 2). Replace mattress
2. Position slightly down from the head of the bed. When the person using the handle comes to a sitting position at the edge of the bed the handle should be right next to them. *The handle should not be positioned to hinder them from getting in and out independently.*
3. Reverse the handle to determine which direction is best.

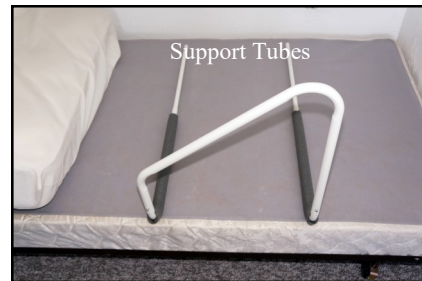

(Figure 1)

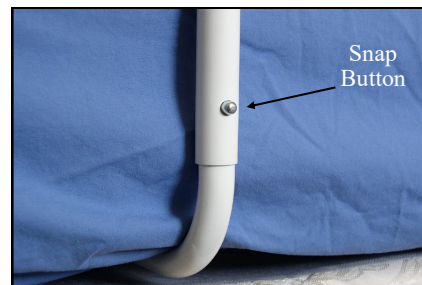

(Figure 2)

Caution:

There should be no more than a 2" space between the mattress and the SlantRail. Reposition if the gap between the mattress and SlantRail becomes greater than 2" to prevent entrapment.