


Freedom Assist Handle Bedrail


Item 510



Freedom Assist Handle on Bed

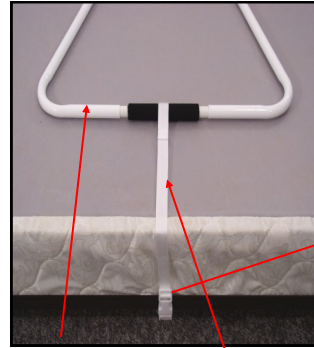


Support tube w/Strap

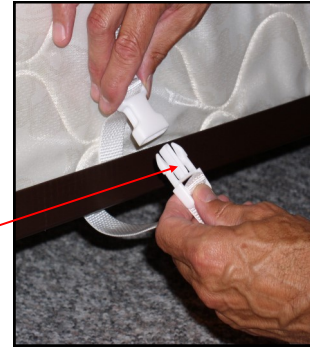

Securing strap buckle

1. Remove mattress and place support tube on top of box spring with strap pulled to the far side of the bed. The strap should be secure to the bed frame . The receiving tubes that the handle inserts into should be tight to the side of the mattress as shown in images .

2. Insert handle legs into receiving tubes. Depress snap buttons to allow leg to pass through receiving tubes.

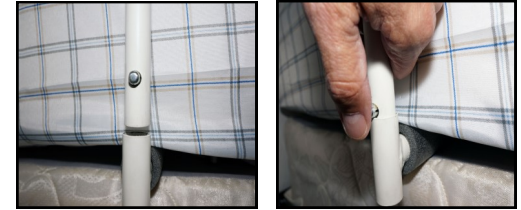

3. Now insert handle leg into support foot with the multiple holes. This will be how you will adjust the height of the handle to best meet your needs. Lock snap button into appropriate hole.

Caution: Read warning below. Securing strap must be secured to the opposite side of the bed as to keep the handle tight to the bed. The handle should not have more than 2 inches between the mattress and the handle legs.

Warning:

This product is not recommended for individuals with cognitive problems (difficulty in understanding their surroundings). If you are taking medications that affects your balance or your ability to sit or stand you should consult with a physician before using this product. If the product is not properly used, entrapment, serious injury or death could result. This product is not intended to carry the full weight of an individual. The product is only intended to assist in bed mobility and rising to a sitting position. These instructions must be carefully followed to insure that the product is properly assembled, installed and used for the stated purpose.

If you have any questions or concerns please contact MTS at 1-800-854-4687. Not for use on hospital style beds.

