

SECURITY HOME BED RAILS





Installation Instructions for single sided and double sided Bed Rail models

For Models 1885 18”/ 5085 30” Double Sided

1. Remove mattress to expose box spring. (**Figure 1**)
2. Cut the tie-wraps that hold the receiving tubes to the side rail. Swing receiving tubes away from the side rail so that the snap buttons snap into the holes in the receiving tubes. Place one side rail on either side of the bed.
3. Use the two center tubes to connect the two side rails. Insert the smaller receiving tube into the barrel of the larger center tube.
4. Important, telescope these tubes into each other until both side rails fit snug to the sides of the mattress to prevent patient entrapment. Make sure that the snap buttons lock into place. (See cautions and Warnings)

Caution: By pulling one side of the bedrail away from the bed, make sure there is no more than a 2 inch spacing between the bed rail and the mattress once snap buttons are locked into place. If there is more than a 2 inch gap, re-adjust snap buttons. If this does not close the gap do not use the Security Bed Rail. There should not be more than 3 inch spacing at the bottom side of the Security Rail and the top of the mattress. If you have a very thin mattress this may be a concern that would need to be addressed. See Warnings.



5. **Figure 2** - shows the side rail in the lowered position. When raising or lowering the side rail, depress the snap button allowing the side rail to move up or down to desired position. When in the proper position the snap buttons will click and lock into the hole in the receiving tube
6. **Figure 2 Insert** - shows the snap button locked in place.
7. To Lower, depress the snap buttons and then lower the side rail. With the rails lowered you can easily make the bed. Your bed cover will hide the bed rails when not in use.

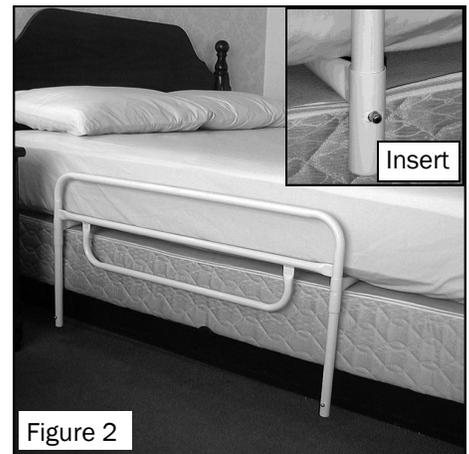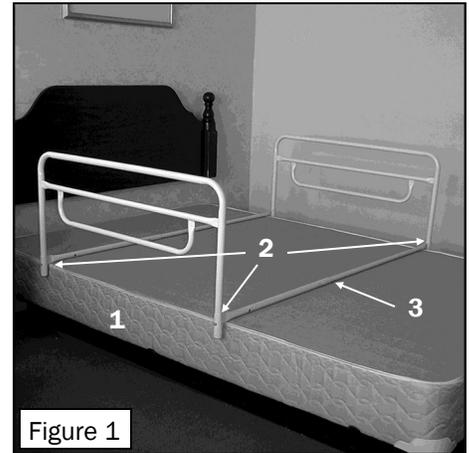

For Models 1875 18” / 5075 30”- One Sided

1. Remove mattress to expose box spring. (**Figure 3**)
2. Cut tie wraps and swing receiving tubes away from the side rail so that the snap buttons snap into the holes in the receiving tubes. Receiving tubes need to be installed on the legs of the bed rail. (**Figure 3 +4**) Single sided models have a “U” shaped support base with foam.
3. Insert the receiving tubes into the connecting “U” shaped support base allowing the snap buttons to lock into place.
4. Place the support base on box spring and position. Ideal positioning is so the patient can get in and out of bed independently (**Figure 3**).
5. Replace Mattress

Caution: When placing bedrail into position on the bed, make sure there is no more than a 2 inch spacing between the Security rail and the mattress side once positioned. If there is more than a 2 inch gap, Reposition. If this does not close the gap do not use. Contact Customer Service.

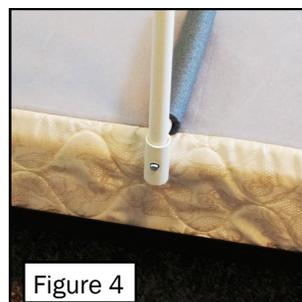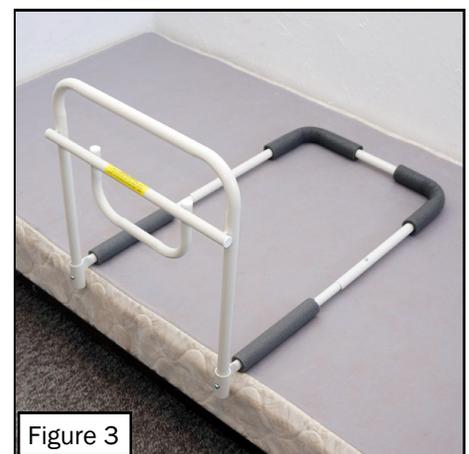

Model Number	Style	Rail Length
1875	Single	18”
1885	Double	18”
5075	Single	30”
5085	Double	30”